

Ti Base for Dental Implants Prosthesis

Precision

20 years working in
digital solutions for the
Dental World



Dental CAM
DC RESEARCH TECHNOLOGIES

Digital Solutions
Implant Library for **EXOCAD**
All Connection



DCA_DentalCAM-IO-exocad-libraries


1-Download Zip **DCA_DentalCAM-IO-exocad-libraries** to your desktop

2-Unzip with 7-Zip

3-Open the folder **DentalCADApp/library/implant** in your Exocad installation

4-Open folder **DentalCAM-IO-exocad-libraries /implant**

5-Copy existing folders into 4 and paste in 3

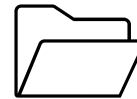

DCA_DentalCAM-IO-exocad-libraries folder

DentalCADApp/library/implant



1-Descargue el Zip **DCA_DentalCAM-IO-exocad-libraries** a su escritorio

2-Descomprima utilizando 7-Zip

3-Abra la carpeta **DentalCADApp/library/implant** en su instalación de Exocad

4-Abra la carpeta **DentalCAM-IO-exocad-libraries /implant**

5-Copie las carpetas de 4 en 3



DCA_DentalCAM-IO-exocad-libraries folder

DentalCADApp/library/implant

