

Ti Base for Dental Implants Prosthesis

Precision

20 years working in
digital solutions for the
Dental World




Dental CAM
DC RESEARCH TECHNOLOGIES



Digital Solutions
Library for EXOCAD - Models
All Connection

DentalCAM-Lab-Analog-exocad-libraries

- 1 Download Zip **Models MicroFit** to your desktop
- 2 Unzip with free 7-Zip
- 3 Open the folder **DentalCADApp/ library/modelcreator/implants** in your Exocad installation
- 4 Open folder **Models MicroFit**
- 5 Copy existing folders into 4 and paste in 3

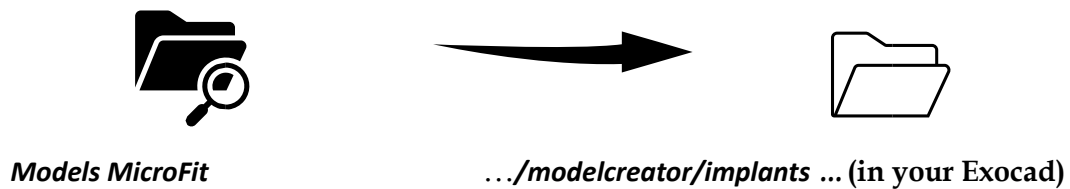

DentalCAM-Lab-Analog-exocad-libraries

- 1 Descargue a su escritorio el Zip **Models MicroFit**
- 2 Extraiga usando 7-Zip gratuito
- 3 Abra la carpeta **DentalCADApp/ library/modelcreator/implants** en su instalacion de Exocad
- 4 Abra la carpeta **Models MicroFit**
- 5 Copie todas la carpetas de 4 en 3

