

# Ti Base for Dental Implants Prosthesis

Precision

20 years working in  
digital solutions for the  
Dental World




**Dental CAM**  
DC RESEARCH TECHNOLOGIES



Digital Solutions  
Library for EXOCAD - Models  
All Connection

DentalCAM-Lab-Analog-exocad-libraries

- 1 Download Zip *DCA\_DentalCAM-Lab-Analog-exocad-libraries* to your desktop
- 2 Unzip with free 7-Zip
- 3 Open the folder *DentalCADApp/ library/modelcreator/implants* in your Exocad installation
- 4 Open folder *DentalCAM-Lab-Analog-exocad-libraries/ modelcreator/ implants*
- 5 Copy existing folders into 4 and paste in 3

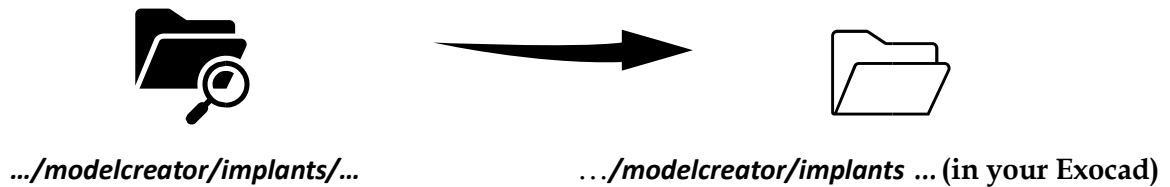

DentalCAM-Lab-Analog-exocad-libraries

- 1 Descargue a su escritorio el Zip *DCA\_DentalCAM-Lab-Analog-exocad-libraries*
- 2 Extraiga usando 7-Zip gratuito
- 3 Abra la carpeta *DentalCADApp/ library/modelcreator/implants* en su instalacion de Exocad
- 4 Abra la carpeta *DentalCAM-Lab-Analog-exocad-libraries/ modelcreator/ implants*
- 5 Copie todas la carpetas de 4 en 3

